# STIMULAN®

Bullet Mat and Introducer

An innovative design for cases with an infected site

**STIMULAN** is a truly absorbable calcium sulfate, specifically designed to complement your dead space management.<sup>1</sup>

✓ Cleared for placement directly
at the site of infection

The innovative **STIMULAN**Bullet Mat and Introducer simplify delivery into hard-to-reach defects.
The Bullet Mat molds long bullets with a groove down one side to reduce the risk of pressurizing the defect site.

The streamlined, flexible Introducer can be inserted deep into the void area and the obturator enables you to push bullets into the surgical site. Bullets can be prepared in two diameters – 7mm and 9mm for varying void sizes.

Streamlined, flexible design that simplifies the delivery of STIMULAN







## Preparation and technique guide for **STIMULAN** Rapid Cure with the **STIMULAN** Bullet Mat and Introducer

#### Choose Bullet and Introducer sizes to suit surgical site

Bullet diameter	Outside diameter of Introducer	Minimum reamed diameter	End-to-end length	Order code
7mm	9mm (black handle)	IOmm	640mm	- 660-001
9mm	llmm (silver handle)	I2mm	480mm	

#### Prepare STIMULAN Rapid Cure 20cc

Use the mixing solution from the tube provided in the pack. Do not add any other fluid. Replace the bead mat with the Bullet Mat.





Empty **STIMULAN**Rapid Cure powder
into the sterile mixing
bowl provided





Add mixing solution and mix thoroughly until a smooth paste is formed (approx. 30 secs)





Apply to Bullet Mat immediately





Allow paste to set undisturbed for at least 8 minutes after mixing. Flex Bullet Mat to release bullets

### Technique for pre-loading and using the Introducer






Ensure retaining tab at the distal tip is as shown





Remove obturator and insert bullets to level required to fill void



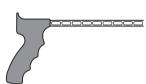

Transfer pre-loaded Introducer to surgical site. Keep horizontal





Insert Introducer slowly into surgical site





Insert obturator and apply positive pressure.

Maintain positive pressure and pull the Introducer back, leaving STIMULAN bullets inside the void

References: I. Biocomposites Ltd, Stimulan Instructions for Use

For indications, contraindications, warnings and precautions see Instructions for Use. Concurrent use of locally administered antibiotics may affect setting time. This brochure may include the use of STIMULAN or techniques that go beyond the current clearance/approval granted by the relevant regulatory authority. Please contact your local representative for further information.

©2018, Biocomposites, STIMULAN, Bringing Calcium to Life and Power to Transform are trademarks/registered trademarks of Biocomposites Ltd.

All rights reserved. No unauthorized copying, reproduction, distributing or republication is allowed unless prior written permission is granted by the owner,
Biocomposites Ltd.

