


STIMULAN®

Bullet Mat and Introducer

Innovation for cases
with an infected site

Streamlined, flexible design
that simplifies the delivery
of STIMULAN®

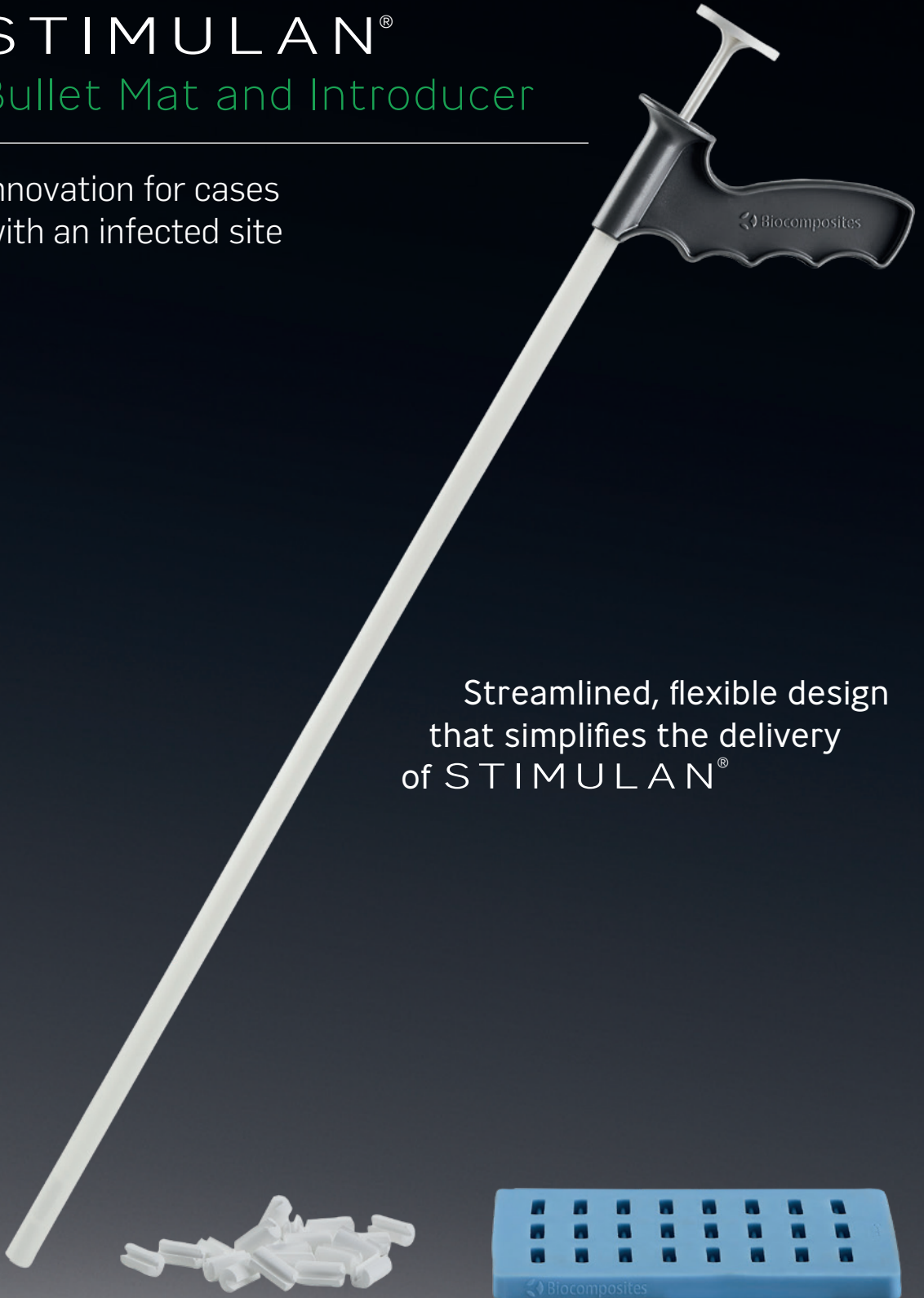

STIMULAN[®]

Bullet Mat and Introducer

Quickly and easily fill hard-to-reach defects

STIMULAN is a truly absorbable calcium sulfate, specifically designed to complement your dead space management.!

- ✓ can be placed directly into infected and non-infected bone voids and defects
- ✓ may help to improve surgical outcomes and lower overall costs of care

The innovative STIMULAN Bullet Mat and Introducer simplify delivery into hard-to-reach defects. The Bullet Mat molds long bullets with a groove down one side to reduce the risk of pressurizing the defect site.

The streamlined, flexible Introducer can be inserted deep into the void area and the obturator enables you to push bullets into the surgical site. Bullets can be prepared in two diameters – 7mm and 9mm for varying void sizes.



Streamlined, flexible design that simplifies the delivery of STIMULAN for deep defects – more efficiently and cost-effectively than using paste

Bullets

7mm

9mm



STIMULAN[®]

Preparation Technique


Choose Bullet and Introducer sizes to suit surgical site

Bullet diameter x length	Outside diameter of Introducer	Minimum reamed diameter	End-to-end length	Order code
7mm x 20mm	9mm (black handle)	10mm	640mm	660-001
9mm x 20mm	11mm (silver handle)	12mm	480mm	

Prepare STIMULAN Rapid Cure 20cc in accordance with the Instructions for Use. Replace the bead mat with the Bullet Mat.


Technique for pre-loading and using the Introducer

1

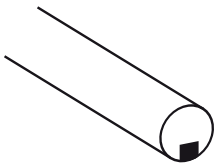


Ensure retaining tab at the distal tip is as shown

2

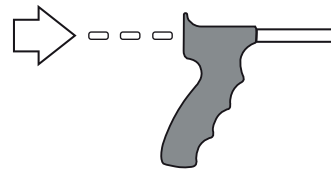


Remove obturator and insert bullets to level required to fill void

3

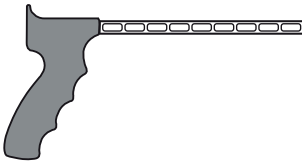

Transfer pre-loaded Introducer to surgical site. Keep horizontal

4

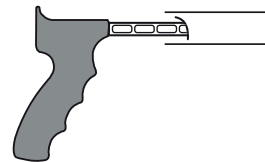

Insert Introducer slowly into surgical site

5



Insert obturator and apply positive pressure. Maintain positive pressure and pull the Introducer back, leaving STIMULAN bullets inside the void



Biocomposites Companion


Your essential guide to making the most of STIMULAN
- all in one straightforward app.

Our accessible app provides all the information you
need to meet different clinical demands when using
STIMULAN.

From surgeons' tips to how-to videos, this is the
expert support tool you need, right at your
fingertips.

- ✓ How to prepare: step-by-step videos
- ✓ Setting times: simple overview
- ✓ Top tips: tried and tested advice for surgeons
- ✓ FAQs: common questions answered
- ✓ Product range: what's available

To download your
Biocomposites Companion,
simply scan the QR code.

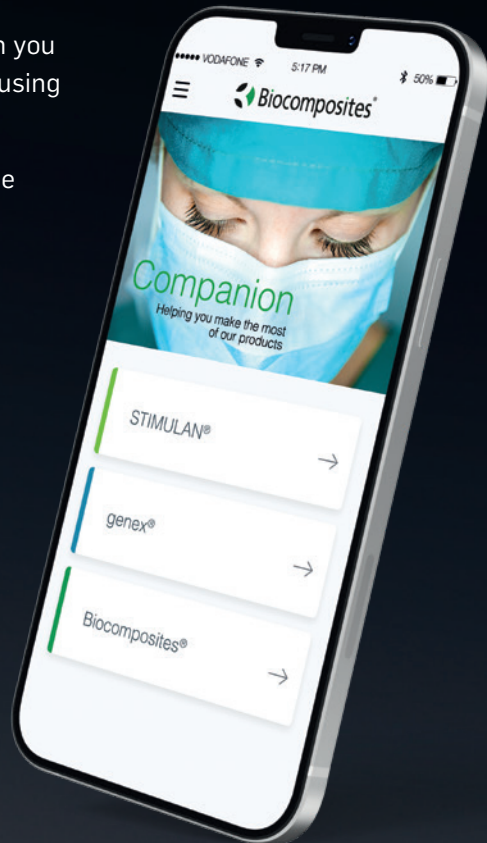

References: 1. Biocomposites, STIMULAN Instructions for Use.

For indications, contraindications, warnings and precautions see Instructions for Use. Concurrent use of locally administered antibiotics may affect setting time.

This brochure may include the use of STIMULAN or techniques that go beyond the current clearance / approval granted by the relevant regulatory authority. Please contact your local representative for further information.

©2024, Biocomposites, STIMULAN, Bringing Calcium to Life, Power to Transform and DRy26 are trademarks/registered trademarks of Biocomposites Ltd. All rights reserved. No unauthorised copying, reproduction, distributing or republication is allowed unless prior written permission is granted by the owner, Biocomposites Ltd.

Patents granted: GB2367552, EP 1204599 B1, US 6780391, EP 2594231 B1, US 8883063, CN ZL201210466117.X, GB2496710, EP 3058899 B1, US 10390954, US 10,588,748, CN ZL201610089710.5

Patents pending: GB1502655.2, GB1704688.9, EP 18275044.8, US 15/933936, CN 108619579A